VK701

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Intended use

Adapters of the VK701series are intended for the connection to the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST series. They are suitable for testing extension cables and device power supply cables for continuity, crossed wires, and for measuring the insulation and PE conductor resistance.

For testing with the VK701 adapter, the

- UNIMET® 800ST/810ST requires operating software version 2.0 or higher and
- UNIMET® 1000ST/1100ST requires operating software version 3.1 or higher.

Function

The automated test with UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST includes checking that the wiring is correct and verifying the connection between the individual pins on the plug and socket. The insulation resistance of the wires in relation to the PE conductor (PE) is also tested.

UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST can print out a test protocol on a connected printer. The test data can be stored in the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST and transferred to a PC software.

General safety information

This manual is intended for qualified personnel working in electrical engineering and electronics! In addition to this manual, the documentation of the device includes a sheet entitled "Important safety instructions for Bender products".

Device-specific safety information

Risk of fatal injury from electric shock!

VK701 are exclusively intended for the connection to the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST test system for testing extension cables and device power supply cables. Never use the adapters for other purposes.

Failure to comply with this safety information may cause electric shock to personnel. Furthermore, substantial damage to the electrical installation and destruction of the device can occur.

Also refer to the manual of the test system "UNIMET...".

Connection

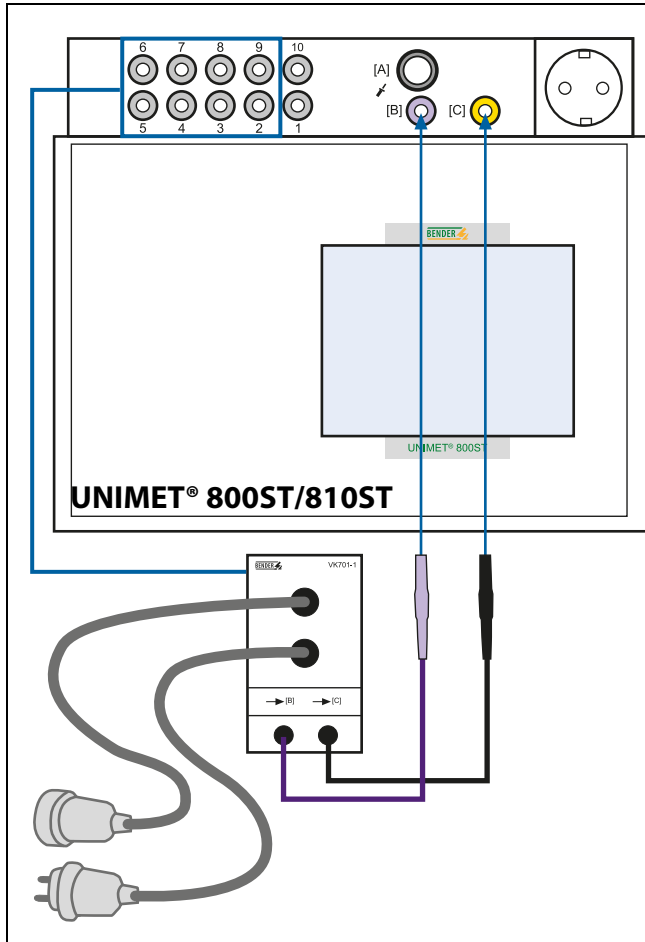
Risk of fatal injury from electric shock!

Prior to connecting the adapter and prior to working on the adapter connections, make sure that no measurement is being started with the test system.

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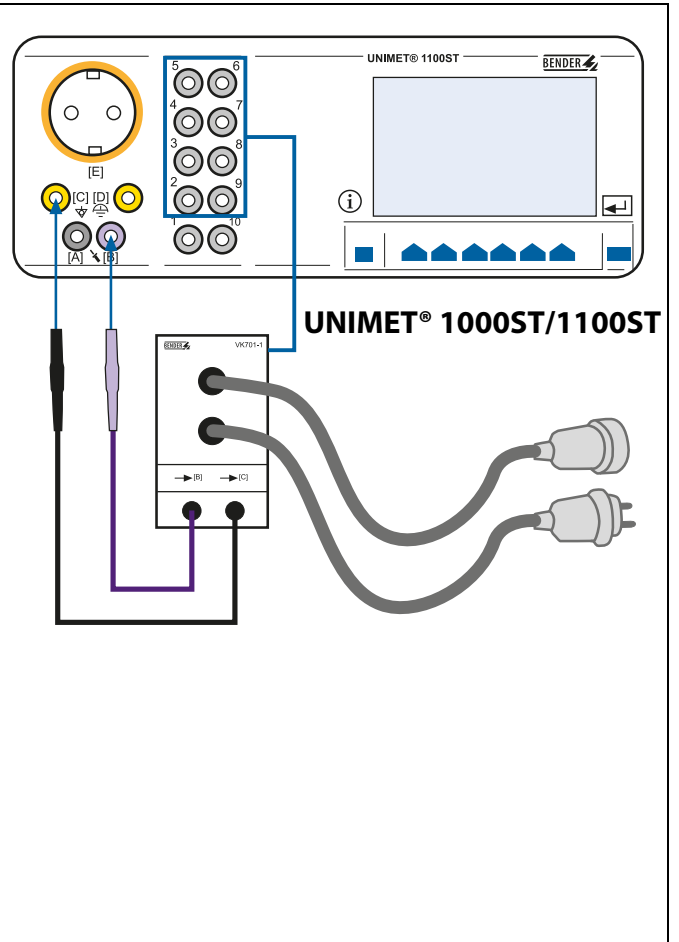
Test

Connection

1. Put the test system onto an even surface. The test system either should be in the switched-off mode or you should be in the main menu.
2. Plug the VK701 adapter and its test cables into the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST system, as described above (connections 2...9, C and B).
3. Connect the DUT to the VK701 adapter.

Zero balancing of the adapter

Prior to first use, the VK701 adapter must be zero balanced.

For this purpose, select the following at UNIMET® 1000ST/1100ST: SYSTEM ADMINISTRATION => SETUP => MORE => test probe, VK701 => test probe => PASSIVE test probe...

At UNIMET® 800ST/810ST select:
System control => Zero calibration PE resistance => Adapter VK701 #108.

Follow the instructions that appear on the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST display. This zero balance will remain, even if the test system is switched off. Before changing to other adapters or measuring cables, these have to be zero balanced again.

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Now the measurement can be started. It should be noticed that in the single test mode the measuring current flows continuously until the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST interrupts the measurement for cooling down.

How to classify a test with VK701

1. Select the VDE 0701-1/VDE 0702-1 test standard.
2. Select "Power supply cord test" resp. "Power supply cord".
3. Now select "VK701" and the test appropriate to the VK701 that is being used. Then enter the length of the cable to be used.
4. Select the type of visual inspection.
5. Save your settings in the "Type catalogue" resp. under "Test procedures".

Use the VK701 as follows

During testing the VK701 adapter pins are live. Don't touch!

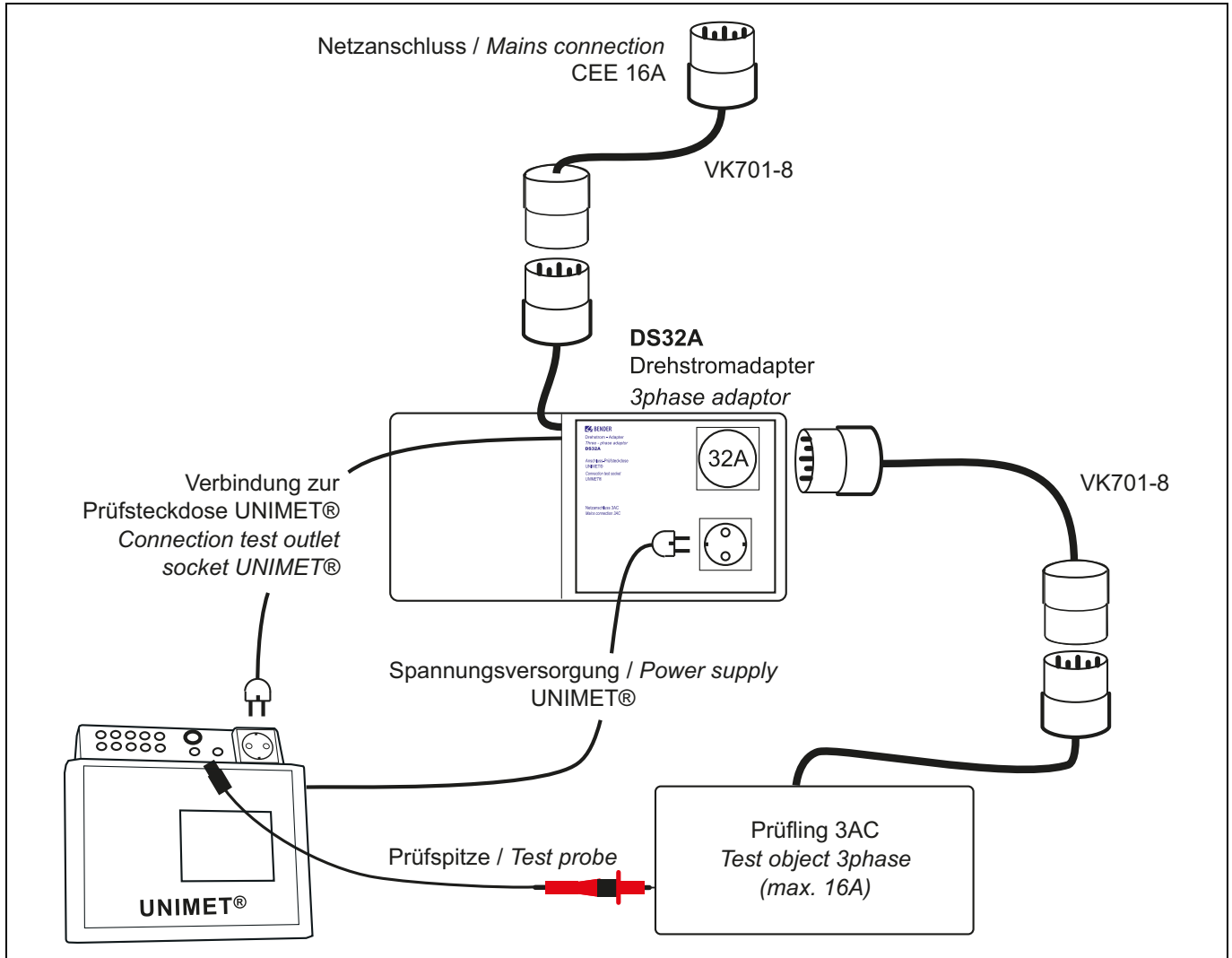
1. Select the respective test from the "Type catalogue" resp. from the "Test procedures".
2. After the visual inspection, the UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST checks that the VK701 connection is correct. Then the electrical test will be started.
3. The test result can be displayed or transferred to a PC software.

Single test

If a fault has been found during the automatic test, the single test can be used for closer examination of the DUT. Select the respective test steps (e.g. "Continuity" or "Conductors reversed").

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The adapter VK701-8 is intended to be used only for the measurement of 16 A three-phase devices in connection with the three-phase adapter DS32A. Observe the instructions described in the data sheet of the DS32A.

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Technical data VK701-1 ... VK701-7

(in combination with UNIMET® 800ST/810ST resp. UNIMET® 1000ST/1100ST)

Insulation coordination acc. to IEC 60664-1

Rated voltage AC 500 V
 Rated impulse voltage/pollution degree 4 kV/3

Test voltage/insulation resistance

Test voltage/insulation resistance DC 500 V
 Test current max. 2.5 mA
 Measuring range 0.04 MΩ ... 299.99 MΩ

PE conductor test

Test current AC >10 A
 Measuring range 0.001 Ω ... 29.99 Ω

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All rights reserved. Reprinting and duplicating only with permission of the publisher. Subject to change!

© Bender GmbH & Co. KG

Photos: Bender archives.

BENDER Group